



**The Most COMFORTABLE
Nose Wash in the World**

INTRO

Nasopure is an innovative natural nasal wash designed by an ENT physician to provide a safe and comfortable cleansing of nasal passag.



Uses

- 1 Allergy Relief**
 - Removes pollen, dust, pet dander, and other allergens from the nasal passages
 - Reduces sneezing, congestion, and post-nasal drip
- 2 Cold & Flu Support**
 - Clears excess mucus during colds
 - Helps relieve nasal congestion and sinus pressure
- 3 Sinus Health**
 - Prevents sinus infections by flushing bacteria and irritants
 - Supports recovery from chronic sinusitis or after sinus surgery
- 4 Moisturizing & Comfort**
 - Hydrates dry nasal passages caused by dry climates, air conditioning, or heating
 - Reduces irritation from dryness or environmental pollutants
- 5 General Nasal Hygiene**
 - Daily nasal cleansing for people exposed to dust, smoke, or pollution
 - Maintains overall nasal cleanliness and reduces risk of infection

Extra Tip: It's safe for adults and children (with proper supervision) and can be used daily for prevention or symptom relief.



Each Breath is Precious!
Nasopure in your routine.

How to use

What makes Nasopure special compared to other nasal washes (like neti pots or regular squeeze bottles) are these key points:

- 1 Ergonomic bottle design – angled to fit naturally in the hand and nose, making it easier and more comfortable to use.
- 2 Controlled flow – the soft, flexible bottle lets you control pressure and flow better than a neti pot (less mess, more comfort).
- 3 Gentle but effective – it rinses the nose without the “drowning” sensation some people feel with traditional methods.
- 4 Pre-measured saline packets – ensure the correct salt balance (isotonic or slightly hypertonic), reducing irritation or burning.
- 5 100% natural wash contain pharmaceutical buffered sodium chloride (NaCl)

1 Prepare the Solution

- Use the pre-measured saline packet that comes with Nasopure.
- Mix it with distilled, sterile, or previously boiled and cooled water.
- Make sure the solution is fully dissolved.

2 Position Yourself

- Lean over a sink.
- Tilt your head slightly forward and to one side.

3 Insert the Tip

- Place the Nasopure bottle tip gently into the upper nostril (the one closer to the ceiling when tilted).
- Make sure it fits comfortably without forcing.

4 Squeeze the Bottle

- Slowly squeeze the bottle to let the saline flow into the nostril.
- The solution will flow through the nasal passages and out the other nostril.

5 Repeat for the Other Side

- Tilt your head the opposite way.
- Repeat the process on the other nostril.

6 Finish Up

- Gently blow your nose to remove remaining solution and mucus.
- Clean the bottle and tip with warm water and let it air dry.

Tips for best results:

- Use lukewarm water for comfort.
- Use daily during allergy season, cold, or sinus congestion.
- Always use safe water to prevent infections.



for more information



for order scan here

Contact us

+971 50 736 5109

www.alafqpharma.com

info@alafqpharma.com

